

## USER MANUAL

### JUGULAR VEIN PRESSURE SCALE

**STEP 1-** Position the patient in cardiac position (lying supine with the bed inclined at 45°)

**STEP 2-** Simply slide the horizontal gauge housing over the vertical scale

**STEP 3-** Locate the sternal angle (Angle of Louis) and place the rubber padded base of the vertical scale over it

**STEP 4-** Easily identify the highest point of the jugular venous pulse using the light source

**STEP 5-** Raise the horizontal gauge and align its red line indicator with the highest point of the jugular venous pulse and ensure that the spirit level is perfectly horizontal.

**STEP 6-** Take the reading off the vertical scale.

